LAMPENWECHSEL

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150 Watt Tungsten Halogen Light Source

WARNING

- Never look directly at lamp when on.
- Modulamp must be firmly in place to operate. Do not override interrupt switch.
- Lamp and surrounding surfaces may be hot!

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OPERATION:

1. Power switch to "OFF" position.
2. Plug power cord into IEC connector on back of unit.
3. Plug power cord into power source per specifications above IEC connector.
4. Firmly tighten captive thumbscrew on Modulamp Assembly.
5. Insert compatible Fiber Optic component into light source adapter.
6. Tighten thumbscrew to hold Fiber Optic in place.
7. Power switch to "ON".
8. Adjust intensity of lamp using control knob.
9. Power switch to "OFF" to shut off light source.

LAMP REPLACEMENT:

1. Power Switch to "OFF."
2. Remove Fiber Optic component.
3. Allow Lamp to cool.
4. Loosen Modulamp thumbscrew until head springs forward.
5. Pull top of Modulamp and lift out of recess. Do **NOT** pull assembly beyond length of cable.
6. Push down lamp holder release lever to remove lamp.
7. Gently unplug ceramic lamp socket.
8. Insert new lamp into socket. Do **NOT** touch bulb or inside of reflector.
9. Push ceramic socket into lamp until flush with base.
10. Slide Modulamp into Light source, "ears" over bottom edge first.
11. Firmly tighten thumbscrew.

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